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## **User manual Senso® Mat** **60cm x 40 cm and XL 120 x 60 cm**

### **Congratulations!**

**You are now the owner of a Senso® Mat, a product “made in Germany”.**

**This sports tool was produced in compliance with valid EU norms, it contains no heavy metals and is made of 100% recyclable plastic.**

The special mat for high demands. Almost 7.000 Senso knobs on the **Senso® Mat XL** make body awareness easier during yoga and meditation. Suitable as an overlay for a massage bed, training mat, smooth floors and much more.

This classic Senso Mat is now available with the new size 60 cm x 40 cm (almost 2.000 Senso knobs). Use this small but nevertheless generously studded mat in your group sessions (Health and fitness) and it will awaken the senses of the participants through the soles of their feet. You can get the Senso effect in your step classes and dance lessons –so simple, save and uncomplicated.

### **Warning**

- Avoid storage in direct sunlight or near heat sources, because otherwise the Senso Mat might deform
- Avoid contact with sharp and pointed objects (e.g. broken glass, pointed rocks in shoe soles) because they can damage the Senso Mat. Do not throw the mat!
- Use sport shoes, regular shoes or be barefoot while training on the Senso Mat.
- Prior to any use, the Senso Mat should be checked for damage because otherwise a safe handling is not guaranteed
- The Senso Mat has to rest on a horizontal, stable and dry ground, to make sure, that the Mat cannot slide away during the exercises.

### **Equipment details:**

The Senso® Mat is made of high-grade Rulon.  
Size: 60 cm x 40 cm or 120 cm x 60 cm

### **Care:**

Use mild standard detergents and disinfectants. Dissolvers as well as acidic or corrosive cleansers can harm the material. The Senso Mat is waterproof.