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## **Manual and Maintenance Instructions Dynair® Ballkissen® Cardo®**

**Congratulations on your purchase of the Dynair® Ballkissen® Cardo®.**

**With the Dynair® Ballkissen you have chosen a TOGU brand product made in Germany.  
The material Ruton is odourless, latex-free and made of 100% recyclable plastic.**

Take your seat and bring yourself automatically in a good and pleasant position!

The adaptable air cushion allows controlled dynamic sitting through the square shape, especially for the beginning of back training and for building the trunk muscles suitable. Similar to the Dynair Ballkissen, but much less shaky - for beginners and seniors.

### **Equipment:**

One side with a velvety smooth surface, the other with gentle senso knobs.

### **Equipment details:**

The Dynair® Ballkissen® Cardo® is made of high quality Ruton and features a needle valve.

Diameter: approx. 36 cm

Load: approx. 200 kg

### **Temperature resistance:**

The Dynair® Ballkissen® Cardo® is resistant against ultraviolet rays and heat. It can be used in temperatures up to 45°C. Please do not store it permanently exposed to direct sunlight (e.g. near a window). The material becomes softer with increasing heat, therefore accelerating the building up of the saddle effect. This tool also resists below freezing temperatures, however it should not be stored in temperatures below -10°C permanently.

### **Care:**

Use any mild standard detergents and disinfectants. Dissolvers as well as acidic or corrosive cleansers can harm the material. The Dynair® Ballkissen® Cardo® is waterproof.

### **How to inflate:**

The Dynair® Ballkissen® Cardo® comes with the right air charge at delivery.

You should take care that the surface is always one level (planar).

If the need arises you can still regulate the air charge with a needle pump (e.g. the TOGU pump or any other regular Soccer Ball pump).

Please pull up the valve between thumb and index finger, in order to avoid puncturing the lower part of the Ballkissen with the needle.

Always moisten the needle of the pump before carefully inserting it into the needle valve.

### **Storage:**

Please always store your Dynair® Ballkissen® Cardo® in a horizontal position.