# HICO-ULTRASONAT 810

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The compact unit of the HICO-ULTRASONAT 810 meets the high requirements of a diversely qualified device with a design and operation which is quite accessible for those not familiar with the technology.

### Function

The **HICO-ULTRASONAT 810** works with a max. frequency of 1.7 MHz and a HF output capacity of approx. 32 W. The electrical energy is converted to mechanical energy through the piezo-ceramic transducer. The resulting vibrations reduce the liquid to the smallest particles of 0.5 to 6 µm in the nebulizing chamber. The transducer requires approx. 15 sec. after the unit starts for automatic calibration.

By automatic calibration of the transducer and a large-volume nebulizing chamber it is possible to nebulize over 500 ml liquid in succession without additional containers. This corresponds to 3-4 hours nebulization without refilling system or up to 8 hours with reduced intensity. An electronic dry run protection device replaces a float switch subject to failure.

An optimal hose guide makes it possible to guide the ventilation hose under the housing. Only the aerosol hose on the device is visible. The nebulizing chamber is surrounded by a gutter for the drainage of dripping water.

A coarse air and a bacteria filter (with CE mark) ensure germ-free mist to guaranty the highest possible hygiene safety.

The aerosol liquid can also be heated with a heating system built into the aerosol hose (as accessory). A heater wire covered with a plastic insulation is embedded into the aerosol hose. The heater only functions when the hose connector has been plugged into the corresponding standard jack on the device and only when the machine is nebulizing, i.e. in function.

### Indications

The **HICO ULTRASONAT 810** is used in the clinical and health care treatment in:

- surgery, pre- and post-operative to prevent infections (pneumonia);
- intensive care, inhaled air humidification;
- internal medicine – specific and non-specific pulmonary diseases, pneumonias, asthma-bronchial bronchiectasis, acute, chronic and super-infected bronchitis;
- paediatrics – mucoviscidosis, pseudocroup, pneumonias, bronchitis;
- ear, nose, throat – inhaled air humidification (tracheotomy), laryngitis;
- balneology – due to a refined dosage, improved application and efficiency in a classic indication.

in diagnostic applications:

- Pulmonology – irritant aerosol application

### Operational safety

The unit meets the requirements of directive 93/42/EEC, is classified as class IIa as a medical product and is manufactured according to the high quality standard ISO 9001.

### Warranty

The HICO-ULTRASONAT 810 conforms to EU directive 93/42/EEC and carries the CE mark. The device was manufactured and tested in accordance with ISO 9001, EN 13485 and EN 60601-1.

For the HICO-ULTRASONAT 810, the warranty of 24 months applies to materials and fabrication.

Figure 2: Standard device with the possibility of permanent nebulization

Figure 3: Unit that can be attached to wall bars

Figure 4: Control panel

### Technical specifications:

Supply voltage*	220-240 VAC / 110-120 VAC	50/60 Hz
Power consumption		approx. 85 W
Current consumption	approx. 0.4 A / approx. 0.8 A	
Nebulizing capacity	max. approx. 4.5 ml/min.	
HF output capacity	approx. 32 W	
Transducer frequency	up to approx. 1.7 MHz	
Particle size	approx. 0.5-6 µm	
Mist temperature	max. 43 °C	
Ambient temperature	10 -30 °C	
Storage temperature	10 - 40 °C	
Dimensions (W x H x D):	300 x 250 x 300 mm	
Chamber volumes	max. 625 ml	
Total height (with undercarriage)	approx. 1200 mm	
Weight (unit)	approx. 4 kg.	
Weight cpl.	Approx. 18 kg	
Classification	Protection class I, Type B; IP X1	

(Subject to changes)  
\*Customized setting in the machine by the manufacturer.  
Note type plate!

### Note:

The **HICO-ULTRASONAT 810** meets the requirements of classes A and B in conformity with EN 55011. In class B, there can be impeded reception with a secondary transmission in the area of 1.67 MHz (outside of the medium waveband) when operating short wave receiving stations in direct proximity. An additional network filter especially designed for the **HICO ULTRASONAT 810** for this case is available. This filter adapted to the **HICO ULTRASONAT 810** is not suitable for other units and has the following specifications:

Nominal voltage: AC 110V / 230V

Nominal current: max. 2A

Internal fuse: Fine-wire fuse 5x20mm  
– 2A neutral

### Order number:

HICO-ULTRASONAT 810 (Standard)	390001
Accessory for	
- Permanent atomisation	390010
- For mobile Ultrasonat (standard device)	390011
- Wall bar system	392900
- Cone-cups for drug nebulization (100 pieces)	382727
- Heatable aerosol hose	383502