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- 1 Control unit with sliding switch
- 2 Indicator light
- 3 Connection cable

#### Explanation of symbols

Do not use the heated blanket when it is folded!

Do not puncture the heated blanket

Not suitable for children under 3 years!

Only use the heated blanket indoors!

The heated blanket may be washed by hand according to the instructions on the label!

Do not bleach!

Do not dry the heated blanket in a tumble drier!

Do not iron the heated blanket!

Do not dry clean!

**IMPORTANT!**  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes provide useful additional information on the installation or operation.

Protection category II

**LOT** LOT number

Manufacturer

## GB Safety instructions

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

- Before connecting the appliance to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Check the heating blanket carefully before each use, if you notice wear or damage
- Do not use if you notice wear, damage or signs of improper use on the electric blanket, the switch or the cables.
- If the power supply cable is damaged, it may only be replaced by **medisana**, an authorised dealer or by qualified personnel to avoid any risk of danger.
- Only use the heating blanket for its intended use as in the instruction manual.
- Do not use the device on children or persons who are disabled, asleep or insensitive to heat (persons who cannot react on overheating).
- Children below the age of 3 years are not allowed to use this device, because they are not able to react on overheating.
- The device must not be used by young children over the age of 3 years, unless the control unit has been adjusted accordingly by its parents or legal guardians or unless the child has been well instructed how to use the control unit properly.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be made by children without supervision.
- The heating blanket may not be operated while folded, creased or placed around the mattress.
- During use, the control switch and lead must not be placed on or under the heating blanket or be otherwise covered in any way.
- No safety pins or other pointed or sharp objects should be attached to the heating blanket or stuck into it.
- Never leave the heating blanket on when unattended.
- The device is only for private use and is not to be used in hospitals.
- Never handle a heating blanket that has fallen into water. Unplug it from the mains outlet immediately.
- Do not use the heating blanket in a damp or humid environment or if the heating blanket itself is damp. You may only use it again once it is completely dry.
- Never expose the control switch or leads to moisture of any kind.
- Keep the mains lead away from hot surfaces.
- Never carry, pull or twist the heating blanket by the mains lead and do not allow the lead to become tangled or snagged.
- Do not repair the heating blanket yourself in the event of damage or malfunction. Only have repairs carried out by an authorized specialist dealer or suitably qualified personnel.
- The heating blanket may only be operated with its special control unit.
- When storing the appliance, allow it to cool down before folding. When storing, do not place any objects on the electric blanket to prevent buckling.

## Health tips and recommendations

- Should you have any concerns about health issues, consult your doctor before using the heating blanket.
- Should you experience persistent pains in the muscles and joints, please inform your doctor.
- Persistent pains can be a symptom of serious illness.
- If you experience any pain or discomfort while using the unit, stop using it immediately.

## Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the it and contact your dealer or your service centre.

The following parts are included:

- 1 **medisana** Cosy heating blanket **HB 675** with control unit
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**  
**Please ensure that the polythene packing is kept away from the reach of children!**  
**Risk of suffocation!**

## Use

With the **medisana** heating blanket **HB 675** you can gently warm and relax your whole body. Snuggle up under the extra-soft heating blanket for a pleasant, cosy well-being. The heating blanket features an electric temperature control that regulates precisely the warmth-supply according to the chosen setting (1-2-3-4). The two outer layers of the three-layer blanket are made of fluffy microfibre fleece. The easy-care material and the detachable operating element make the blanket hand-washable up to 30°.

## Operation

Plug the unit into the mains outlet and move the slider control **1** from position **0** to position **1**. Operation indicator lamp **2** initially lights up green, then it flashes red and subsequently it lights up red for the remainder of the warmup phase. After a few minutes you will be able to feel the heating blanket getting noticeably warmer. When the electric blanket reaches the set temperature, the lamp **2** lights up green again. When the next warm-up phase begins, operation indicator lamp lights up red again. If you require a higher temperature setting, move the switch **1** to the next positions, **2** or **3**, or to the maximum heat setting, position **4**. Each time switch is moved, lamp lights up green and then flashes red according to the level at which the switch is set (two, three or four flashes). The lamp then lights up again constantly in red for the remainder of the warm-up phase. Should the heating blanket get too warm for you, slide the switch **1** back to a lower setting. To switch off the device, return the switch to position **0**. The red power LED **2** will go out, indicating that the unit is switched off. Now unplug the heating blanket from the mains outlet.

The heating blanket is designed in such a way that settings **1** and **2** are suitable for continuous use. Push the switch to position **1** or **2** if you want to use it in continuous operation. The appliance is equipped with an anti-overheating device. The heating blanket will switch off automatically after approx. 3 hours of continuous use. To switch it back on, first move the switch **1** back to position **0** then back to the desired heat setting. Unplug the heating blanket from the mains outlet when it is no longer in use.

## Cleaning and maintenance

- Before washing the heating blanket, unplug it from the mains outlet and let it cool down.
- Unplug the control switch unit from the heating blanket using the connector on the underside.
- The electric blanket may be washed by hand according to the instructions on the label. Never immerse the control switch unit in water as moisture must not be allowed to get into the unit.
- To dry, lay the heating blanket flat on a moisture-absorbent towel or mat. When the heating blanket is completely dry, it should be reconnected to the control switch unit in the correct manner. The heating blanket may only be operated with its special control unit.
- When storing the appliance, allow it to cool down before folding. Store the heating blanket in a clean and dry place. When storing, do not place any objects on the electric blanket to prevent buckling.

## Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

## Technical specifications

Name and model: **medisana** Cosy heating blanket **HB 675**  
 Power supply: 220 - 240V~ 50/60 Hz  
 Heat output: approx. 120 W  
 Automatic switch off: after approx. 3 hours  
 Switching levels: 0 - 1 - 2 - 3 - 4  
 Dimensions: approx. 200 x 150 cm  
 Weight: approx. 1.7 kg  
 Mains lead length: approx. 2.3 m  
 Item number: 60231  
 EAN number: 40 15588 60231 3

**In accordance with our policy of continual product improvement, we reserve the right for technical and design changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

## Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for **medisana** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**medisana GmbH**  
 Carl-Schurz-Str. 2  
 41460 NEUSS  
 GERMANY

The service centre address is shown on the attached leaflet.

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