

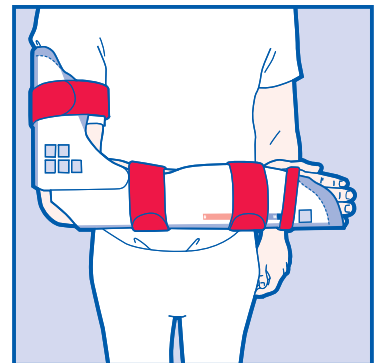
Delta-XpressTM

- (DE) Gebrauchsfertige Ellenbogenschiene
(EN) Ready to use elbow splint system
(FR) Système d'attelle de coude prêt à l'emploi
(ES) Férula para codo lista para usar
(PT) Ortótese de cotovelo pronta a utilizar
(IT) Ortesi per gomito pronta per l'uso
(NL) Gebruiksklare elleboogspalk
(SV) Bruksfärdig armbågsskena
(NO) Bruksklar albueskinne
(DA) Albueskinne, "Klar-til-brug"
(FI) Käyttövalmis kyynärpäälasta
(ZH) 即用型肘关节夹板系统
(JA) すぐに使える肘用副木
(AR) جهاز جبيرة المرفق الجاهز للاستخدام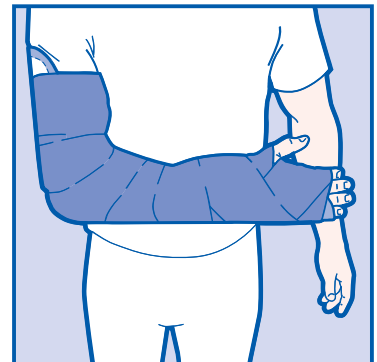

Delta-Xpress™


1.

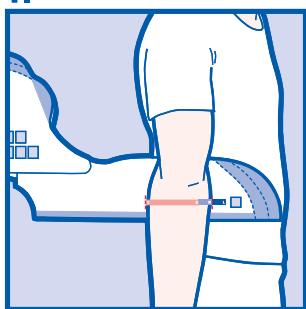2.


3.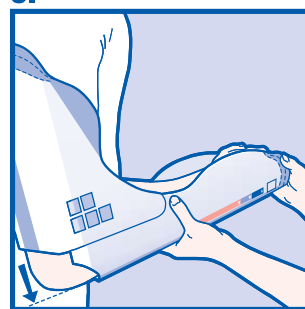


4.


Optional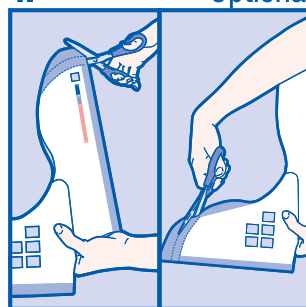

5.1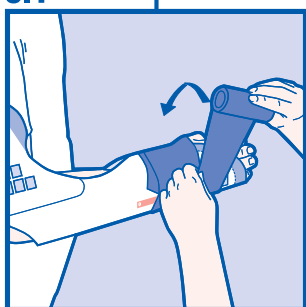


5.2

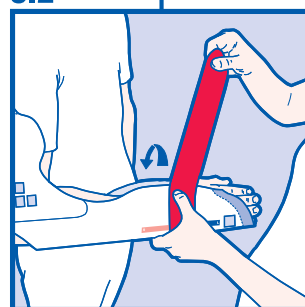
6.1


6.2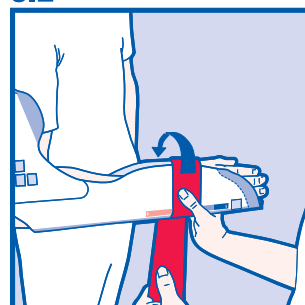

7.1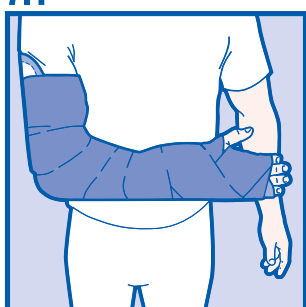

7.2

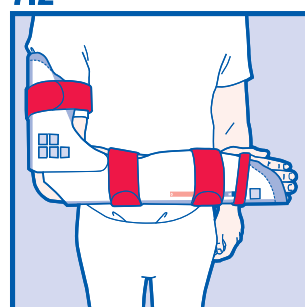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Ready to use elbow splint system

Description

Delta-Xpress™ is a ready to use splint system for the immobilization/stabilization of the elbow joint. The pre-shaped splint system can easily be adjusted to the patient's body size and shape. The splint is secured to the limb with a bandage or with hook and loop fasteners. If fixed with a bandage, it is recommended to use a cohesive bandage. Because of its pre-shaped, anatomical form, the splint is fast and easy to apply. It is made of lightweight material that offers required immobilization/stabilization.

With its 100 % x-ray translucency, Delta-Xpress™ can be kept on the limb during x-ray examination.

Indications

Delta-Xpress™ is used for the short-term immobilization/stabilization of the elbow joint after surgery or after injuries of the arm such as fractures, wounds, sprains, strains, dislocations and burn injuries.

It is particularly recommended for

- Post-traumatic emergency immobilization/stabilization
- Primary care of soft tissue injuries
- Post-operative immobilization/stabilization
- Temporary immobilization/stabilization of the elbow before synthetic cast, synthetic splint or Plaster of Paris is applied
- Temporary immobilization/stabilization during X-ray examination.

Precautions

Any wound in the splinted area should be dressed prior to application of the splint. If necessary, use additional padding to cover bony prominences and prevent pressure points.

It is recommended not to use the product at temperatures higher than 40° C.

It is recommended not to use the product at a relative humidity higher than 75 %.

Application Instructions

The term and use of the product is the responsibility of a qualified medical professional.

1. The width of the lower arm indicates which size of splint to use. Measure size of the arm by holding the size scale on the elbow splint to the widest part of the lower arm (fig. 1).
2. Choose the correct size of splint. Fold splint open for easier application to the limb (fig. 2).
3. Check length of splint by holding it in the intended final position. In the correct final position the elbow is bent in a 90° degree angle and the splint has full contact to the upper arm and to the lower arm (fig. 3). The space between the distal end of the splint and the distal end of the small finger should be 2 fingers wide at minimum. The space between the proximal end of the splint and the axilla should be 2 fingers wide at minimum.
4. Adjust the length of the splint by cutting the distal and/or proximal side of the splint with standard scissors (fig. 4)
5. If desired, apply a stockinette on the limb and/or padding on areas of bony prominences to avoid pressure points.
6. Position splint on the limb as described under application step 3 (fig. 3) and secure with bandage or hook and loop fasteners.