
TAS® Toe Alignment Splint

- EN** Toe Alignment Splint
- DE** Hallux Valgus Zehenorthese
- ES** Férula para juanete
- FR** Attelle pour hallux valgus
- IT** Ortesi correttiva per alluce valgo
- NL** Hallux valgus-teenspalk
- DA** Hallux valgus tåskinne
- SV** Tåskena för hallux valgus
- PL** Szyna korekcyjna na paluch koślawy
- SL** Opornica za izkrivljen nožni palec (hallux valgus)
- HR** Odstojnik za nožne prste za Hallux Valgus
- PT** Tala de dedo para hallux valgus

+DARCO


TAS® (Toe Alignment Splint)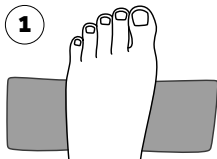

EN Metatarsal band | **DE** Metatarsalband



EN Toe loop
DE Zehenschlaufe

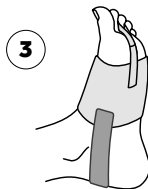EN Heel band
DE Fersenband

EN Toe loop | **DE** Zehenschlaufe

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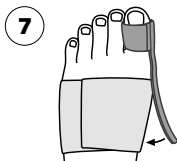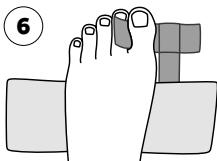TAS® (Toe Alignment Splint) | Toe Alignment Splint

Important notes

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your doctor, specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities. Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation, etc.) please tell your doctor.

Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This device assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility.

Do not wear the product in direct contact with injured, irritated, damaged or broken skin. This device is not recommended for direct skin contact. You should use bandages, for example. While wearing this device, ensure sufficient forefoot clearance and avoid all kinds of outside mechanical influences. You must not drive a vehicle while you are wearing this product.

Note: This device is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses.

Purpose

Night positioning orthosis for correcting or realigning toe deformities.

Indications / Contraindications

Indications

For conservative and postoperative correction / realignment of toes in:

- Hallux valgus
- Tailor's bunion
- Hammer toes

Contraindications

No contraindications are known.

Design Features

The TAS® toe orthosis is a bandage for alignment that is used instead of taping. It enables correction or fixation and alignment of the toes in the desired position. A strap at the back of the foot reduces slippage of the midfoot bandage. The soft material makes it comfortable to wear.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Use

Note: The orthosis is worn without shoes and is not suitable for walking.

Hammer toes (fig. 1 – 3): The elastic metatarsal band is designed so that it can be adjusted to the foot individually. Place the band around the foot as shown (fig. 1 – 2). The heel strap prevents the bandage from slipping and is put on as shown in fig. 3.

Tailor's bunion (fig. 4 – 5): The small toe loops are fixed to the metatarsal band according to need using the hook and loop strap (fig. 4 – 5).

Hallux valgus (fig. 6 – 9): The big toe loop is fixed as shown in fig. 6 – 7. The stabilising element can be inserted into the big toe loop (fig. 8 – 9).

Technical data

Size: Universal fit for right and left

Colour: White

Material

Elastic strap: Polyester, polyamide, viscose, elastane

Hook strap: Polyamide

Fleece strap: Polyamide

Velour: Polyamide

Label: Polyester satin

Label: Polyester satin

Stabilising element: Polyethylene, polyester

Sewing thread: Polyester

Combination with other products

No information

Maintenance

The product does not require maintenance.

Care instructions

- Hand wash: Cold or lukewarm, max. 30°C
- Do not bleach
- Do not dry in dryer
- Do not iron
- Do not dry-clean

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

TAS® (Toe Alignment Splint) | Hallux Valgus Zehenorthese

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ALBO-Healthcare GmbH