MELAcontrol® Helix

Gebrauchsanweisung
Instructions for use
Инструкция за употреба
Brugsanvisning
Kasutusjuhend
Käyttöohjeet
Notice d'utilisation
Οδηγίες χρήσης
Használati útmutató
Istruzioni per l'uso
Naudojimo instrukcija
Gebruiksaanwijzing
Instrukcja używania
Instruções de utilização
Instrucțiuni de utilizare
Bruksanvisning
Návod na použitie
Navodila za uporabo
Návodem k použití
使用说明

Interster International B.V.
Samsonweg 2
1521 RC Wormerveer
The Netherlands
www.interster.nl

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Intended use

MELAcontrol Helix is a test body system (Process Challenge Device = PCD) in accordance with EN ISO 11140 for products with narrow lumen. The test body system is used to demonstrate the rapid and complete penetration of steam into medical devices that are sterilized in a steam sterilizer with type “B” cycles. The chemical indicator contained in the sealing cap is a type 2 indicator according to EN ISO 11140-1. If the indicator is exposed to a defined combination of time, steam and temperature, it shows a defined colour change from blue to pink: at 134 °C for at least 3:30 min and at 121 °C for at least 15 min. If all the air has been removed from the sterilization chamber, steam enters the test body system and the indicator shows a uniform colour change.

MELAcontrol Helix supplements the physical measurements and is only suitable for testing the following steam sterilizers:

- steam sterilizers with type “B” cycles in accordance with EN 13060
- steam sterilizers with type “S” cycles in accordance with EN 13060, of which the scope of supply covers the reprocessing of products with narrow lumen
- Large steam sterilizers with a chamber volume of up to 250 l in accordance with EN 285

Symbols on the product

Manufacturer of the product

Date of manufacture of the product

Distributor of the product

Can be used for steam sterilization

Observe user manual or electronic user manual

Hollow body load


Usable until

Article number of the product

Batch designation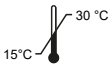

Temperature range storage


Do not use if packaging is damaged

Not for reuse (indicator strip)

Keep dry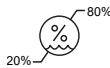

Humidity range storage

Keep away from sunlight

Application

Comply with the following for safe handling:

- Allow the MELAcontrol Helix to acclimatise to the required ambient temperature (15-30 °C) before the first use.
- Only use dry, cooled test body systems and, if possible, those that have been blown out with compressed air. Residue condensate in the test body has a negative effect on air removal.
- Insert an unused indicator strip in the plastic bracket of the sealing cap before every test.
- Make sure that the hose protrudes from the housing.



1

2


3

4


Unscrew the cap from the helix tube.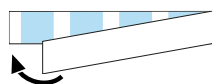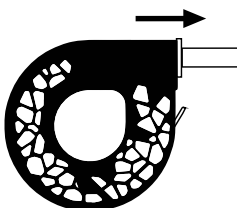

Insert the indicator strip, fold first, as far as possible into the helix tube.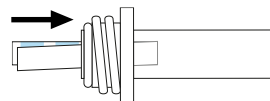

Screw the cap hand-tight onto the helix tube.

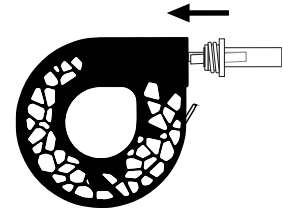⚠ CAUTION**Danger of burns from hot condensate!**

Position the test body for sterilization in a level position, otherwise hot condensate can run out of the test body after removal.

- Let the test body cool before removal.

Use for daily routine check

1. Place a new indicator strip in the test body.
2. Slide the test body with the flat side facing downwards onto a tray in the empty sterilization chamber.
3. Start the steam sterilizer in a validated sterilization program according to the intended use.
4. Remove the test body from the steam sterilizer when the program run has ended.
5. Remove the indicator strip from the sealing cap. Evaluate the indicator strip.

Use as batch control

1. Place a new indicator strip in the test body.
2. Place the test body in the sterilization chamber above the load, with the flat side facing down.
3. Start the steam sterilizer in a validated sterilization program according to the intended use.
4. Remove the test body from the steam sterilizer when the program run has ended.
5. Remove the indicator strip from the sealing cap. Evaluate the indicator strip.

Evaluation**Sterilization successful**

If the indicator strip has completely changed to a colour from the gradient, air removal and steam penetration are sufficient.

Documentation

The evaluation of the results must be performed by a sufficiently trained person holding responsibility for this task. The colour change is caused by an irreversible chemical reaction. If stored correctly, the indicator strips can be archived for years. EN ISO 15882 only requires a description of the results. This can be performed in written or digital form. Comply with the valid national and regional regulations.

Important information regarding routine operation

Replace the test body after 250 indicator strips have been used.

Please comply with the current recommendations issued by the Robert Koch Institute (RKI) and the information contained in DIN 58946-7. Further information regarding routine operation is to be found in user manual of the steam sterilizer.

Storage

Store MELAcontrol Helix at 15-30 °C and 20-80 % humidity and protected from excessive dust, air flow, humidity and direct sunlight. Store unused indicator strips in the sealed PE bag. If these conditions are observed, unused indicator strips have a shelf life of 4 years from the date of manufacture.

Incorrect storage of the indicator strips can cause unwanted colour changes. Further information regarding storage is provided on the packaging. A secure determination of malfunctions in the process can only be guaranteed up to the specified date of expiry.

Disposal

Dispose of this product and spare parts, e.g. seals, that are no longer used properly. Comply with all relevant disposal specification in terms of possibly contaminated waste.

The packaging protects the product against transport damage. The packaging materials have been selected for their environmentally-friendly and recycling properties and can be recycled. Returning the packaging to the material flow reduces the amount of waste and saves raw materials.

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