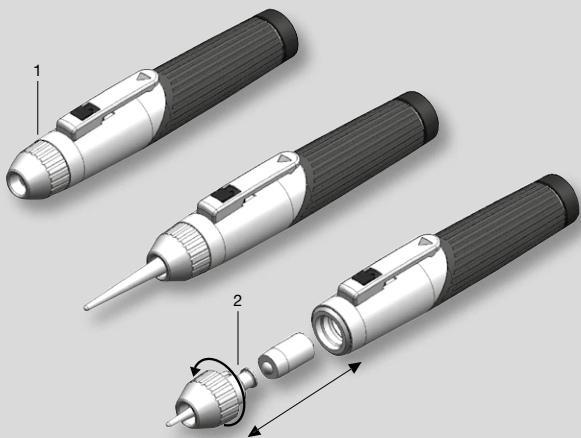
☛ **HEINE® mini-c**
Cliplamp and Ear Light

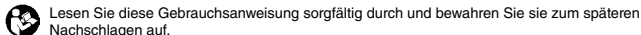
HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
med 3906 2013-11-28

HEINE® mini-c Cliplamp and Ear Light

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
SUOMI	17
PORTUGUÊS	19

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HEINE® mini-c Cliplamp and Ear Light

ENGLISH

Please read and follow these instructions for use and keep them for future reference.

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

For U.S. only:

Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Intended Use

- HEINE® mini-c Cliplamp (1):

The HEINE® mini-c Cliplamp is designed solely for the examination of the surface of the skin and the anterior segment of the eye.

- HEINE Ear Light (2):

The HEINE Ear Light in combination with the HEINE® mini-c Cliplamp is exclusively designed for the lighting of the transition area from the auricle (external ear) to the external acoustic meatus (ear canal).

Hygienic Cleaning / Processing

The HEINE® mini-c Cliplamp is not suitable for treatment in the ultrasonic bath, for sterilisation, spray disinfection or treatment by immersion in liquids. The surface of the HEINE Ear Light may be damaged by alcoholic cleaning agents.

- Manual Cleaning and Disinfectant wipes:

Use a damp cloth for cleaning the HEINE® mini-c Cliplamp and HEINE Ear Light. For cleaning and disinfection, a disinfectant agent suitable for plastic medical products must be used. Follow the instructions of the manufacturer of the disinfectant for this purpose.

- Inspection / Functional testing:

The HEINE Ear Light must be checked for its functionality for its intended use on a regular basis. The item must be discarded, if any damage is visible or suspected.

Maintenance

This product does not require regular maintenance.

Service

This product does not require regular service.

Disposal

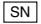
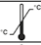
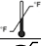
 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

General Notes and Warnings

-  Check the correct operation of the product at regular intervals. Do not use the equipment if it shows visible signs of damage.
 - The HEINE Ear Light is used by hearing aid specialists or trained personnel as a device for the anatomical adjustment of the geometry of hearing aids.
 - The HEINE Ear Light is not suitable for the examination of the eardrum and may not be introduced too deeply into the acoustic meatus.
 - Do not use alkali manganese batteries containing lithium as bulb life will be reduced.
 - The performance of this instrument can only be guaranteed if genuine HEINE Accessories are used.
-  This product is a precise optical instrument. Please handle this product with care and store it in a clean place.
-  This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

	The CE mark indicates that the product complies with the European medical device directive 93/42/EEC.
	Catalogue- or reference number
	Serial number
	Manufacturer
	Date of manufacture
	Product bearing this symbol may not be disposed of together with general household waste, but instead requires separate disposal according to local provisions. (European Waste Electrical and Electronic Equipment Directive, WEEE)
	Temperature limits in °C for storage and transport
	Temperature limits in °F for storage and transport
	Humidity limitation for storage and transport
	Please read and follow the instructions for use and keep them for future reference (background color blue; foreground color white)
	Fragile, handle with care!
	Keep dry
	Interference may occur in the vicinity of equipment marked with the following symbol
	“Grüner Punkt” (country-specific)

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